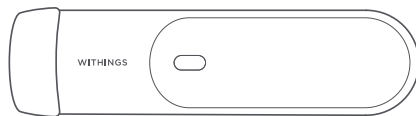
Withings Thermo

QUICK INSTALLATION GUIDE

GUIDE D'INSTALLATION RAPIDE | KURZANLEITUNG INSTALLATION | GUÍA DE INSTALACIÓN |
GUIDA ALL'INSTALLAZIONE | GUIA DE INSTALAÇÃO

Read this manual before use

EN | FR | DE | ES | IT | PT

THANK YOU FOR CHOOSING WITHINGS THERMO

MERCI D'AVOIR CHOISI WITHINGS THERMO | DANKE, DASS SIE WITHINGS THERMO GEWÄHLT HABEN |
GRACIAS POR ELIGIR WITHINGS THERMO | GRAZIE PER AVER SCELTO WITHINGS THERMO
OBRIGADO POR ESCOLHER O WITHINGS THERMO

Need help?


Besoin d'aide ? | Brauchen Sie Hilfe? | ¿Necesitas ayuda? | Ti serve aiuto? | Precisa de ajuda?

➔ withings.com/support/thermo

This guide is available in additional languages at: www.withings.com/guides

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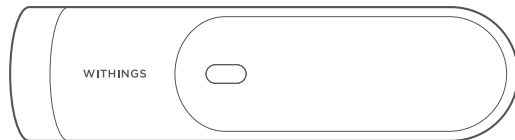How to install your Withings Thermo

Comment installer votre Withings Thermo| So installieren Sie Withings Thermo| Cómo instalar Withings Thermo| 4-5

1

Visit thermo.withings.com from your smartphone or tablet, and follow the instructions to install the Withings Thermo app.

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2

In the Thermo app, follow the instructions to install Thermo.

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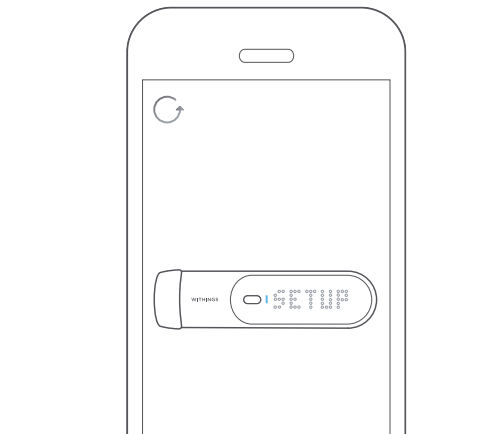
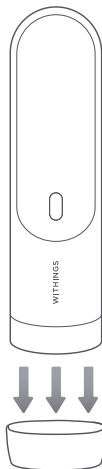
i **Clear any sweat or hair from the forehead before taking a measurement.**

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2

Press the button to turn the thermometer on.

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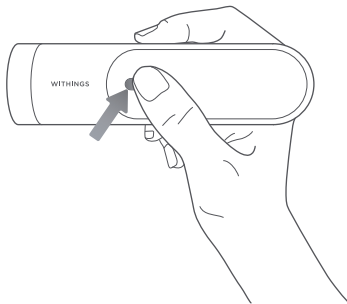
3

Starting from the middle of the forehead, slowly scan straight across to the hairline, up to 1cm over the skin.

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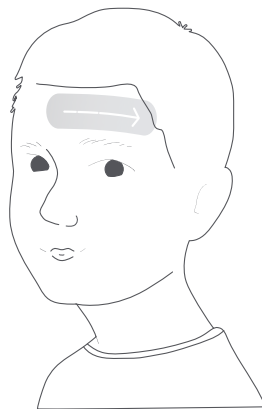
4

Thermo will vibrate twice when the measurement is complete. The value of the temperature is displayed on the device. The unit of this value is either °F or °C. You can change this unit in the Thermo app settings. If you change the unit, make sure to update your device in the Thermo app settings.

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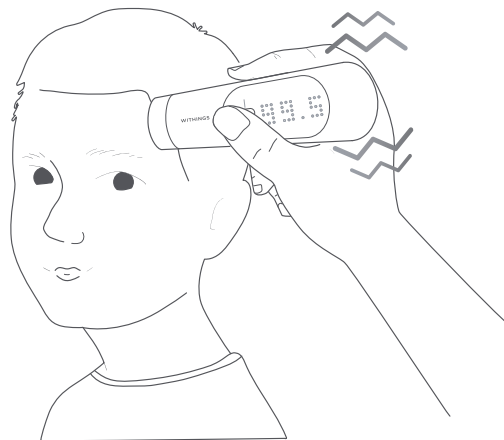
5

To assign the temperature to a user, scroll through names on the touch-sensitive zone (1).

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
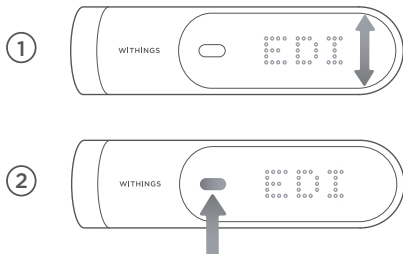
6

A colored LED light indicates the fever level according to the user's age.

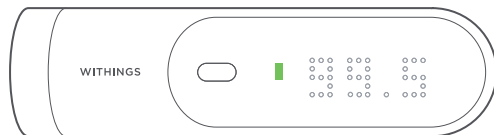
- FR** Une LED de couleur indique le niveau de fièvre en fonction de l'âge de l'utilisateur.
- DE** Die gemessene Temperatur wird über eine farbige LED-Anzeige dargestellt. Dabei wird das Alter des Nutzers beachtet.
- ES** Una LED en color indica el nivel de fiebre teniendo en cuenta la edad del usuario.
- IT** Un indicatore a LED a colori indica il grado di febbre, che tiene conto dell'età del paziente.
- PT** Um LED colorido indica o nível de febre, de acordo com a idade do usuário.

- No fever** | Absence de fièvre | Kein Fieber | Ausencia de fiebre | Assenza di febbre | Sem febre
- Mild Fever** | Fièvre modérée | Erhöhte Temperatur | Fiebre moderada | Febbre moderata | Febre baixa
- High fever** | Fièvre élevée | Fieber | Fiebre alta | Febbre alta | Febre alta

How to take a temperature

Comment prendre une température| So messen Sie die Körpertemperatur| Cómo medir la temperatura |

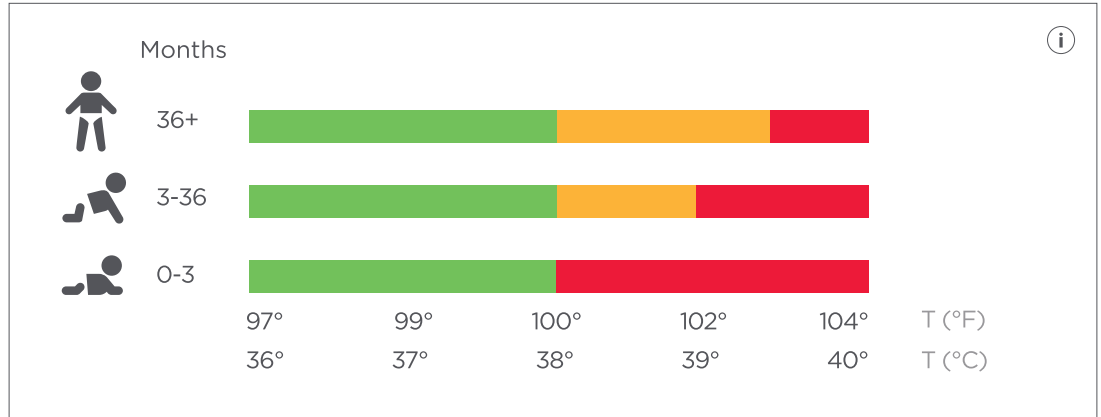Important advice

Recommendations importantes | Vorsichtsmaßnahmen | Consejos de uso | Precauzioni d'uso | Recomendações importantes

The person being measured must be indoors for 15 minutes prior to taking a reading. | The person being measured must have dry skin. If needed, dry skin with a towel prior to taking a reading. | The person being measured and the thermometer should be in the same ambient temperature for at least 15 minutes prior to taking a reading. | The person being measured should not have exercised within 15 minutes prior to taking a reading. | Don't take a measurement while or immediately after nursing a baby.

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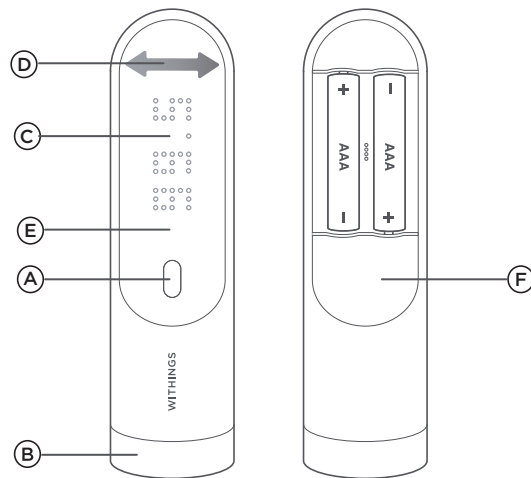
Warranty

Garantie | Garantie | Garantía | Garanzia | Garantia

US Withings SA, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, ("Withings") warrants the Withings-branded hardware product ("Withings Product") against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of **ONE (1) YEAR** from the date of original retail purchase by the end-user purchaser ("Warranty Period"). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or Quick Installation Guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

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UK Withings Two (2) Years commercial guarantee - Withings Thermo Withings SA, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, France («Withings») warrants the Withings-branded hardware product («Withings») against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser («Warranty Period»). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withi

EN Withings SA hereby declares that the device Withings Thermo is in conformity with the essential requirements and other relevant requirements of Directive 2014/53/EU. The declaration of conformity can be found at: withings.com/compliance

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The crossed-out wheeled bin symbol on your product, battery and/or accessories means that the item is classified as electrical and electronic equipment.

QIG_SCT01_63

WITHINGS

Withings Thermo | Smart temporal thermometer - Model: SCT01 - V1.1

© 2020 Withings SA. All rights reserved.