POPme

ITA

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POPme

ENG

Instruction manual - Disposable Face masks

Description

Three-layer: external breathable layer, intermediate layer with high filtering power, internal resistant and hypoallergenic layer. Equipped with hypoallergenic ear tubular elastics and moldable traumatic protective nose clip.

Storage and transportation

The expiry date is shown on the product packaging. Before use, check that the product is still within the stated validity period (expiration date). The product should be stored in a dry, cool and clean place.

5) Use both hands to shape the nose clip so that it fits perfectly on your face. Pressing the nose clip with one hand could result in a worse performance of the device.

6) Place the mask on your face so that your chin is fully covered before accessing a contaminated area.

How to wear the mask correctly

Figure 1
Before wearing the mask, make sure that both your hands are clean.

1) Separate the upper and lower flaps in order to create a shell shape. Slightly bend the center of the nose clip.

2) Make sure both flaps are fully unfolded.

3) Hold the elastic bands one in each hand, with the nose clip facing upward.

4) Place the elastic bands behind your ears and press the nose clip slightly onto your nose.

5) Use both hands to shape the nose clip so that it fits perfectly on your face. Pressing the nose clip with one hand could result in a worse performance of the device.

6) Place the mask on your face so that your chin is fully covered before accessing a contaminated area.

IMPORTANT NOTICE

L10 disclaims any liability, may it be direct or indirect (including, but not limited to, loss of revenues, turnover and / or profits), on information that are included here and provided by L10. The user is responsible for determining the suitability of the product for the aim it is intended to use.

Warnings and restrictions on use

- Always make sure that the product is:

- suitable for the use;

- worn correctly;

- worn throughout the exposition period;

- replaced when necessary.

- An appropriate selection, training for correct use and proper maintenance are essential for the user to have maximum protection from contaminants in the air.

- If instructions are not followed to use these products as a the protection of the mask tract and / or do not wear the product correctly for the entire period of exposure, it may compromise the user's health and cause serious illness or permanent disability.

- To check the adequacy and correct use, please refer to national regulations, follow all the instructions provided in the manual or contact a health and safety expert at work.

- Before use, the user must be trained for the correct use of the complete product, in accordance with current Health and Safety regulations / guidelines.

- These products do not contain components made of natural rubber latex.

- using it with beard, mustache or sideburns that can limit the adhesion of the respirator to the face will compromise its good seal.

- Leave the contaminated area immediately if:

a) breathing becomes difficult;

b) dizziness or other ailments are felt.

- Dispose of and replace the respirator if it is damaged or breathing is too difficult.

- Do not alter, modify or repair this device in any way.

- Do not use in an environment with insufficient oxygen content (<17%).

- Do not use when the concentrations of the contaminant are immediately dangerous to health.

- Do not use in case of escape; in areas with risk of explosion, comply with the relative regulations.

- Leave the work area immediately if:

- breathing becomes difficult

- there is a sense of anguish and dizziness.

- The device must be replaced if it is damaged, if the mask resistance becomes excessive or in any case at the end of the work shift.

This device is not a medical device in the sense of Regulation EU/2017/745 (surgical masks) nor is it personal protective equipment in the sense of Regulation EU/2016/425 (filtering masks type FFP2).

WARNING: Not suitable for children under 2 years. Children wearing face coverings should be supervised at all times.

- SINGLE USE

Face coverings should not be worn by any person who may have breathing difficulties whilst wearing a face covering and/or whilst carrying out strenuous activities which may affect breathing rates.

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